

PLEASE READ CAREFULLY AND KEEP THIS PACKAGE INSERT FOR FUTURE USE IN CASE YOU HAVE A PROBLEM.

BAUSCH + LOMB

**re-
nu**
multi-purpose solution

sensitive™

DIRECTIONS: TO OBTAIN DISINFECTION AND CLEANING THESE STEPS MUST BE COMPLETED.

STEP 1:

Place at least 3 drops of **re nu sensitive** multi-purpose solution on each side of contact lens and rub for 20 seconds. No separate daily cleaner necessary.

STEP 2:

Remove surface debris by rinsing thoroughly with **re nu sensitive** multi-purpose solution.

STEP 3:

Place cleaned contacts in lens case and fill with fresh **re nu sensitive** multi-purpose solution. Soak for at least four hours. Remember to always use fresh solution. Never re-use.

Your lenses are now ready to wear. No saline rinse necessary. If any debris remains on contacts, rinse with **re nu sensitive** multi-purpose solution prior to insertion.

STORAGE: You may store your lenses in the unopened lens case until ready to wear, up to a maximum of 30 days.

TO REMOVE PROTEIN FROM YOUR CONTACTS:

The special combination of re nu sensitive with **ReNu® 1 Step™ Daily Protein Remover Liquid** is a simple and convenient way to effectively remove protein deposits from soft contact lenses on a daily basis. **ReNu® 1 Step™ Daily Protein Remover Liquid** is specially formulated for daily use to simultaneously enzymatically clean and disinfect your lenses every time you soak your lenses in **re nu sensitive**.

Follow directions provided with the enzymatic product and use as directed by your eye care professional.

Note: Proper lens care requires removal of protein deposits from your lenses.

CONTENTS:

A sterile, isotonic solution that contains boric acid, edetate disodium, poloxamine, sodium borate and sodium chloride; preserved with DYMED® (polyaminopropyl biguanide) 0.00005%.

ACTIONS:

Cleans, loosens and removes accumulations of film, deposits and debris from soft contact lenses. Destroys harmful micro-organisms on the surface of the lens. Rinses, stores and rewets lenses before insertion. **For use in combination with ReNu® 1 Step™ Daily Protein Remover Liquid.**

INDICATIONS (Uses):

BAUSCH + LOMB re nu sensitive multi-purpose solution is indicated for use in daily cleaning, protein removal when used with **ReNu® 1 Step™ Daily Protein Remover Liquid**, rinsing, and storage of daily and extended wear soft contact lenses and chemical (not heat) disinfection, as recommended by your eye care professional.

CONTRAINDICATIONS (Reasons not to use):

If you are allergic to any ingredient in this product, do not use.

WARNINGS: PROBLEMS WITH CONTACT LENSES AND LENS CARE PRODUCTS COULD RESULT IN SERIOUS INJURY TO THE EYE.

It is essential that you follow your eye care professional's directions and all labeling instructions for proper use and care of your contact lenses and lens care products, including the lens case. Eye problems, including corneal ulcers, can develop rapidly and lead to loss of vision. Daily wear lenses are not indicated for overnight wear and should not be worn while sleeping. Clinical studies have shown the risk of serious adverse reactions is increased when these lenses are worn overnight. Extended wear lenses should be regularly removed for cleaning and disinfection or for disposal and replacement on the schedule prescribed by your eye care professional. Clinical studies have shown that there is an increased incidence of serious adverse reactions in extended wear contact lens users as compared to daily wear contact lens users. Studies have also shown that the risk of serious adverse reactions increases the longer extended wear lenses are worn before removal for cleaning and disinfection or for disposal and replacement. Studies have also shown that smokers have a higher incidence of adverse reactions. If you experience eye discomfort, excessive tearing, vision changes, redness of the eye, immediately remove your lenses and promptly contact your eye care professional. It is recommended that contact lens wearers see their eye care professional twice each year or, if directed, more frequently.

- To avoid contamination, do not touch tip of container to any surface. Replace cap after using.
- Not for use with heat (thermal) disinfection.

You should fill your lens case with fresh solution every time you store your lenses, and never "top-off" or re-use solution. You should discard your solution immediately after your lenses have been removed from the lens case. You should not expose or store your lenses in or rinse your lens case with any water, such as tap, bottled or distilled, or with any non-sterile solution.

Empty, then clean and rinse your lens case with re nu sensitive multi-purpose solution. Air dry your lens case each time you remove your lenses. In order to permit excess solution to drain, you can flip over your lens case while air drying. Replace your lens case monthly.

Failure to discard solution from lens case after each use or use of water to care for your lenses may lead to contamination resulting in eye injury and potential loss of vision. See accompanying instructions for additional important safety information.

Important Safety Information:

- Always follow the product directions for use. Failure to follow product directions may lead to vision loss.
- Visit your eye care practitioner regularly.
- Always wash and dry hands before handling lenses.
- Do not use tap water, bottled water or saliva with lenses or lens case.
- Only use fresh solution to clean and disinfect contact lenses.
- Always discard any remaining solution in your lens case after each disinfection cycle.
- Saline or rewetting drops will not disinfect your lenses.
- Always replace your solutions, lenses and lens case as directed.
- To avoid contamination, do not touch tip of container to any surface. Replace cap after using.
- Not for use with heat (thermal) disinfection.

PRECAUTIONS:

- Always discard solution from lens case after each use
- Keep the bottle tightly closed when not in use
- Use before expiration date marked on the carton and bottle
- Discard remaining solution within 90 days after opening
- Keep out of reach of children
- Store at room temperature

ADVERSE REACTIONS (Problems and what to do):

The following problems may occur: eyes sting, burn or itch (irritation), comfort is less than when lens was first placed on the eye, feeling of something in the eye (foreign body, scratched area), excessive watering (tearing) of the eye, unusual eye secretions, redness of the eye, reduced sharpness of vision (poor visual acuity), blurred vision, rainbows or halos around objects, sensitivity to light (photophobia), or dry eyes.

If you notice any of the above:

IMMEDIATELY REMOVE YOUR LENSES.

- If the discomfort or problem stops, then look closely at the lens.
- If the lens is in any way damaged, DO NOT put the lens back on your eye. Place the lens in the storage case and contact your eye care professional.
- If the lens has dirt, an eyelash, or other foreign body on it, or the problem stops and the lens appears undamaged, thoroughly clean, rinse and disinfect the lens, then reinsert it.
- If the problem continues, **IMMEDIATELY** remove the lens and consult your eye care professional.

If any of the above symptoms occur, a serious condition such as infection, corneal ulcer, neovascularization or iritis may be present. See immediate professional identification of the problem and prompt treatment to avoid serious eye damage.

GOOD LENS CARE PRACTICES:

- Always wash and rinse your hands before you handle your lenses.
- Clean, rinse and disinfect your lenses each time you remove them.
- Always handle the same lens, the right or the left, first in order to avoid mix-ups.
- After use, always empty and rinse the lens case with fresh **re nu sensitive** multi-purpose solution, and allow to air dry.
- Replace your lens case monthly.

HOW SUPPLIED: **BAUSCH + LOMB re nu sensitive** multi-purpose solution is available for purchase in 4 fl. oz. (118mL), 8 fl. oz. (237mL) and 12 fl. oz. (355mL) plastic bottles. Bottles and cartons are marked with a lot number and expiration date.

Toll Free Product Information

Call: 1-800-553-5340

®/™ denote trademarks of **BAUSCH & LOMB Incorporated**.

© BAUSCH & LOMB Incorporated

Manufactured by: **BAUSCH & LOMB**, Rochester, NY 14609. Made in USA.